

IV3000[◊] Application

Peripheral IV3000 dressing

Suitable for dressing peripheral catheters

Step 1



First remove section 1 from the rounded end of the dressing. This contains 2 strips and a date label.

Step 2



Peel off the strips and place on the catheter wings for extra security.

Step 3



Remove the backing paper marked 2. Position the dressing over the catheter port. Remove the backing papers marked 3 from the non woven sections and adhere over the catheter wings.

Step 4



Using the tab gently peel off the top carrier marked 4.

Step 5



Mould the film around the catheter, ensuring that the dressing is secure and the patient is comfortable. Finally place the date label from section 1 on the dressing.

Removal



Gently peel back the non-woven wings to the point at which they reach the film section. Then take one corner of the film and gently pull parallel to the skin. Release and repeat the process.

Further information

For education on how to use IV3000 dressings, please call Erica Atyeo, Territory Business Manager, M: 07960 740716



IV3000 1 hand ported order codes

S&N Code
4006

NHS Cat Code
ELW046

Dimension
7cm x 9cm

Carton
100