

# IV3000<sup>◇</sup> Application

## Shaped IV3000 dressing

Suitable for dressing jugular, central line, PICC and midline catheters.

### Step 1



Peel off the backing paper marked 1. Place the dressing over the cannula site, ensuring that the entry site is as close to the centre of the dressing (clear section) as possible.

### Step 2



Arrange the ported area around the lumen and remove the backing papers marked 2 from the non-woven wings.

The non-woven wings can be gently pulled together and overlapped to ensure the weight of the lumen is well secured.

### Step 3



Peel off the top film carrier marked 3, using the tab at the base of the dressing, pulling gently towards the direction of the port.

### Step 4



Smooth the dressing into place, paying particular attention to the area around the hub and catheter to minimise the risk of creating 'tunnels' which can allow access to bacteria.

Ensure your patient is comfortable.

### Removal



Gently peel back the non-woven wings to the point at which they reach the film section. Then take one corner of the film and gently pull parallel to the skin.

Release and repeat process.

### Further information

For further information, training or support please contact Erica Atyeo, Territory Business Manager, M: 07960 740716

IV3000 order codes				
	S&N Code	NHS Cat Code	Dimension	Carton
	4006 <sup>†</sup>	ELW046	7cm x 9cm	100
	66004009	ELW112	9cm x 12cm	50
	4008 <sup>†</sup>	ELW054	10cm x 12cm	50